

AMPOULE BREAKER



SAFETY IN OUR HANDS

d.
DANIELS

REDUCE THE RISK OF INJURY WHEN BREAKING AMPOULES. IDEAL FOR WARD, ITU, THEATRE, A&E USE AND FOR MOBILE MEDICAL SERVICES.

AMPOULE BREAKER

Daniels product code: AB700

- Tough polypropylene
- Safe to use
- Easy disposal
- Holds up to 200 ampoule caps
- Flap closes for security between uses
- British patent no. 9910615.5

TO USE:

- Insert cap of ampoule through large hole
- Slide the neck along to locate appropriate size slot
- Invert so fluid collects in bottom of ampoule
- Press down to snap off ampoule cap



Daniels Healthcare Limited

Head Office

72-80 Akeman Street, Tring
Hertfordshire HP23 6AF
Tel: +44 (0)1442 826881
Fax: +44 (0)1442 826880
info@daniels.co.uk
www.daniels.co.uk

Factory

Unit 14, Station Field Industrial Estate,
Kidlington, Oxfordshire OX5 1JD
Tel: +44 (0)1865 371841
Fax: +44 (0)1865 841869
factory@daniels.co.uk



INVESTOR IN PEOPLE



Certificate No. FM 29035

d.
DANIELS