

# SHARPSGUARD® Sharps Container

**REMEMBER! Always  
dispose of sharps at  
the points of use.**

Assemble correctly... snap  
lid on all around firmly  
and fill in label details.

Place on bracket OR take to  
point of use in a POUDS® tray.

Lock door when  
contents reach  
fill line.

Ensure container  
door is in the open  
position before use.

Complete label  
and tag the bin.

Close temporary  
closure when not in  
use.

Remove container and  
dispose of according to  
your local policy.



**ST270**

SHARPSGUARD®

www.daniels.co.uk | © DANIELS HEALTHCARE LTD 2010

Unless otherwise stated, the information contained herein is at the specific request of the user and has been sponsored and made available by Daniels Healthcare Ltd.



Safety, Service and Support Every Day