

SHARPSGUARD® Sharps Container

**REMEMBER! Always
dispose of sharps at
the points of use.**

Assemble correctly... snap
lid on all around firmly
and fill in label details.

Place on bracket OR take to
point of use in a POUDS® tray.

Lock door when
contents reach
fill line.

Ensure container
door is in the open
position before use.

Complete label
and tag the bin.

Close temporary
closure when not in
use.

Remove container and
dispose of according to
your local policy.



ST27P

SHARPSGUARD®

www.daniels.co.uk | © DANIELS HEALTHCARE LTD 2010

Unless otherwise stated, the information contained herein is at the specific request of the user and has been sponsored and made available by Daniels Healthcare Ltd.



Safety, Service and Support Every Day