

SHARPSGUARD® Sharps Container

**REMEMBER! Always
dispose of sharps at
the points of use.**

Assemble correctly... snap
lid on all around firmly
and fill in label details.

Lower lid
between
uses.



Lock lid when
rotary flap
indicates a 'full
warning'.

Place container
on bracket,
securely fitted to
wall or trolley.

Remove container and
dispose of according to
your local policy.

ST28Y

SHARPSGUARD®

www.daniels.co.uk | © DANIELS HEALTHCARE LTD 2010
Unless otherwise stated, the information contained herein is at the specific request of the user
and has been sponsored and made available by Daniels Healthcare Ltd.

d.
DANIELS®

Safety, Service and Support Every Day