

# SHARPSGUARD® Sharps Container

**REMEMBER! Always  
dispose of sharps at  
the points of use.**

Assemble correctly... snap  
lid on all around firmly  
and fill in label details.

Lower lid  
between  
uses.



Lock lid when  
rotary flap  
indicates a 'full  
warning'.

Place container  
on bracket,  
securely fitted to  
wall or trolley.

Remove container and  
dispose of according to  
your local policy.

**ST28P**

SHARPSGUARD®

www.daniels.co.uk | © DANIELS HEALTHCARE LTD 2010  
Unless otherwise stated, the information contained herein is at the specific request of the user  
and has been sponsored and made available by Daniels Healthcare Ltd.

d.  
DANIELS®

Safety, Service and Support Every Day