

SHARPSGUARD®

Sharps Container Assembly

1. Select the correct type and size of SHARPSGUARD® sharps container for your use.
2. Ensure you use the lid that matches the SHARPSGUARD® container you have selected. The lid colour should match the colour of the label. Note that SHARPSGUARD® 0.5, 0.6, com-plus and 1 are supplied ready assembled.
3. Place the lid on the top of the SHARPSGUARD® container so that the aperture is best suited for your use. The aperture does not have to be immediately above the label.
4. With the container resting on a firm surface, press down on one side of the lid with the ball of your hand.
5. Work your way around the SHARPSGUARD® container pushing the remainder of the lid into position. An audible 'click' is normally heard when the final part of the lid is pushed into place.
6. Test the seal on the lid by trying to take the lid off.



7. Complete the appropriate sections of the label.



8. Place the SHARPSGUARD® container on a bracket, tray or in a mobile holder as appropriate.



ST35

POUDS®

A system for the disposal of sharps at the point of use to help reduce the risk of sharps injuries by incorporating simple to use trays kickabouts, holders and brackets to bring the SHARPSGUARD® container range closer to the point of use.



SHARPSGUARD®

www.daniels.co.uk | © DANIELS HEALTHCARE LTD 2010

Unless otherwise stated, the information contained herein is at the specific request of the user and has been sponsored and made available by Daniels Healthcare Ltd.

d.
DANIELS®

Safety, Service and Support Every Day