

SHARPSGUARD®

Sharps Container Door Positions



Door open position

The door in the lid is fully open allowing for disposal of sharps into the container



Temporary closure position

The door in the lid has been pulled across to the temporary closure position. Accidental access to the contents is reduced. The lid can still be opened allowing for further use.

“Temporary Closure”
Moulded within lid



Permanent closure position

The door in the lid is closed and locked firmly by pushing right across. The lid cannot be opened again and the sharps container no longer used.

“Permanent Closure”
Moulded within lid

ST39

SHARPSGUARD®

www.daniels.co.uk | © DANIELS HEALTHCARE LTD 2010

Unless otherwise stated, the information contained herein is at the specific request of the user and has been sponsored and made available by Daniels Healthcare Ltd.



Safety, Service and Support Every Day