

PATSLIDE® Patient Transfer

An essential part of hospital equipment in Wards, Casualty, Operating Theatres, Radiology, anywhere that speed and ease of patient transferring is required.



The drawsheet beneath the recumbent patient is loosened on both sides and the receiving trolley butted against the bed. The person transferring the patient reached across and grasps the drawsheet at the patient's shoulder and pelvic level and slightly tilts the patient towards him/herself. The receiving person slips the edge of the PATSLIDE® Patient Transfer beneath the patient who is then lowered.



The PATSLIDE® Patient Transfer is beneath the drawsheet and is resting partially on both the trolley and the bed, creating a smooth bridge. The patient may now be slid gently from bed to trolley (or visa versa) with minimum patient discomfort.

PATSLIDE® not only makes the job easier and safer, but also reduces the risk of injury to nursing staff when transferring patients. The distress and discomfort that patients can experience when being transferred is eliminated.

PATSLIDE® has proven itself through daily use in hospitals around the world.

This unique product has reduced the incidence of back strain occurring during patient transfer; a factor which previously contributed to many lost working hours and long-term staff disability.

PATSLIDE® is available in the following sizes:

	LENGTH	WIDTH
All Purpose	1525 mm	635 mm
Theatre	1525 mm	560 mm
Slimline	1400 mm	480 mm
Duo-Set (2 Piece)	750 mm	635 mm each

PATSLIDE®

- Safe & easy to use
- Static Insulative
- Lightweight & Resilient
- Radio-Lucent
- Highly Impact Tolerant
- Easy to clean

Manufactured in Australia by: Briggate Medical Company (A division of Drewson International Pty Ltd)
A.B.N. 92 006 609 559 www.briggate.com.au

Briggate Medical Company's authorised representative in the European Community (as defined in Article 14 (2) of the Council Directive: 93/42/EEC) is: Balwood Enterprises Ltd, 30 Vicarage Close, Langford, Biggleswade, Bedfordshire, SG18 9QS England.



PATSLIDE® is NOT a lifting device and should not be used as such. The use of PATSLIDE® in any way other than as described is neither recommended nor approved.

PLEASE REFER TO YOUR OWN MOVING & HANDLING POLICY



Download app
for 24/7 Support



A MAUSER Company

Safety, Service and Support - Every Day, Affordably & Responsibly

The Application of Innovation

www.daniels.co.uk © DANIELS HEALTHCARE LTD 2017

Unless otherwise stated, the information contained herein is at the specific request of the user and has been sponsored and made available by Daniels Healthcare Ltd.